

Infection control process for IQoro® feasibility kits

The process documented here is based upon one identified by an NHS SLT department and is for your consideration. The instructions relating to cleaning are those provided by IQoro®.

How to tell when an IQoro® feasibility kit is clean and ready to use

- The IQoro® devices are provided in a strong re-usable zipbag.
 - Both of the IQoro devices® are in their respective plastic cases inside the zipbag.
 - There is a small plastic disposable bag and a pair of PPE gloves in the zipbag
- A feasibility kit in this state indicates that the contents are clean and ready to use.**

When an IQoro® has been used

- The SLT will put the used device back in its case and then place the case in the additional small plastic bag and knot it – indicating that the devices are ready for cleaning. The gloves should be disposed of appropriately.
- A feasibility kit in this state indicates that the contents are not fit for immediate use.**

Cleaning the IQoro® and returning to use

- The IQoro® devices and plastic cases should be cleaned using washing up liquid or detergent, rinsed and dried (clean paper towel or air dry). They can also be cleaned in the dishwasher if placed on the top rack.
- Once clean and dry, the devices should be returned to their cases and placed in the zipbag along with a fresh disposable plastic bag and pair of gloves. The kit is now complete and ready to be used again.